

Safespaces[®] Hi-Lo Beds

Designed due to popular demand and in association with Royal Masterpeace, we can now offer the Cosyfit mounted on various adjustable bases. We designed them for those who still need the safety of a fully padded bed but also require height adjustable, Trendelenburg features and profiling bed functions. For more information about the Cosyfit please see our website www.safespaces.co.uk

Cosyfit Hi-Lo[®]

The Cosyfit Hi-Lo is a robust hi-lo bed system suitable for those with limited mobility, who thrash, kick, roll, and who may therefore be unsafe in an ordinary hi-lo bed or at risk of falling. It is not suitable for those with a great deal of mobility, those who can stand, climb or pull to standing.

The Cosyfit Hi-Lo is a development of the Cosyfit, a tried and tested sleep system which Safespaces have been providing for those with limited mobility for many years. The Cosyfit Hi-Lo enables improved lifting and handling and better provision of care for clients at a comfortable height above floor level.



Cosyfit Hi-Lo[®] (Profiling)



The Cosyfit Hi-Lo (Profiling) is a profiling bed with higher, softer sides, suitable for those at risk of hurting themselves or falling in most profiling beds, due to headbanging, thrashing or fitting.

It is suitable for those with extremely limited mobility, who would not be able or inclined to try to climb over the raised backrest or knee break.

Cosyfit Hi-Lo[®]

- The Cosyfit Hi-Lo has a fully padded interior, high sides and a mattress base. There is the option for soft pvc doors with a large pvc window on one or both sides.
- The large pvc window/s allow light and prevent the client from feeling enclosed. The client can view and be viewed by their carer in the rest of the room, and can view sensory equipment if required.
- The bed enclosure is safely secured to the lifting mechanism, which eliminates any entrapment risk between the Cosyfit and the Hi-Lo mechanism.
- The Cosyfit Hi-Lo has a variable height option (see Technical Specifications for range) and Trendelenburg and Reverse Trendelenburg functions. These are operated by a handset outside of the Cosyfit, which is fitted with a flexible cord (like a telephone cable). The operator can then be with the client whilst operating the bed functions.

OTHER FEATURES

- The Cosyfit Hi-Lo is suitable for use with ceiling track hoists, and at its lowest position there is still clearance for use of mobile hoists.
- The Cosyfit padded bed enclosure can be removed from the Hi-Lo base and both can be used separately if required. This makes it a highly flexible and versatile piece of equipment, with a range of uses:
 1. As the Cosyfit Hi-Lo
 2. As the Cosyfit freestanding at floor level
 3. As a height adjustable bed base, for use with the Cosyfit mattress as provided, or with another of similar dimensions



WHO IS THE COSYFIT HI-LO FOR?

The Cosyfit Hi-Lo is suitable for those with a limited amount of mobility, who may thrash, roll, kick or fit and therefore may be unsafe in an ordinary bed. It is not suitable for those who can stand, pull to standing or climb.



Cosyfit Hi-Lo[®] (Profiling)

- The Cosyfit Hi-Lo (Profiling) has a fully padded interior, soft sides of up to 76cm and a profiling mattress base. There is the option for a fully opening panel with a pvc window on one or both sides to facilitate caring.
- The Cosyfit Hi-Lo (Profiling) has variable height option (see Technical Specifications for range), back support and knee

break, and Trendelenburg and Reverse Trendelenburg functions. These are operated by a handset outside of the Cosyfit, which is fitted with a flexible cord (like a telephone cable). The operator can then be with the client whilst operating the bed functions.

- The Cosyfit Hi-Lo (Profiling) is suitable for use with ceiling track and mobile hoists.



WHO IS THE COSYFIT HI-LO (PROFILING) FOR?

The Cosyfit Hi-Lo (Profiling) is suitable for those with an extremely limited amount of mobility who need a high level of care, but who may also thrash, roll, kick or fit and therefore may be unsafe in an ordinary profiling bed. It is not suitable for those who can stand, pull to standing or who could climb over the back rest or knee break.

OPTIONS

HEIGHT OF COSYFIT SIDES

- Height of Cosyfit Hi-Lo sides up to 1.2m.
- Height of Cosyfit Hi-Lo (Profiling) sides up to 76cm. We recommend a maximum height of 76cm for the Cosyfit Hi-Lo (Profiling). This is because if the client's mobility would require sides any higher than this, they most likely have the mobility to climb over the back rest or knee break when raised.

It is important for you to assess your client's level of mobility, and decide what height of the Cosyfit sides are adequate to keep your client safe relative to their care needs, and which product is most suitable for them. Please discuss this with us.

- One or 2 opening sides with clear pvc windows.
- Holes provided to your specifications on the opening side/s for gastrostomy, oxygen or other tubes.

TECHNICAL SPECIFICATIONS

- Maximum height of the Cosyfit Hi-Lo sides: 1.2m
- Maximum height of the Cosyfit Hi-Lo (Profiling) sides: 76cm
- Minimum height of bed base (underside of mattress): 40cm, maximum height 70cm
- Mattress thickness on both: 15cm
- Maximum height of whole Cosyfit Hi-Lo at full height extension (1.2m high Cosyfit): 1.95m
- Maximum weight of client (safe working load): up to 120kg (nearly 19 stone). (Other options are available, including Bariatric. Please enquire.)
- Both products have integral cable tidies and twin wheel-locking castors at each end (the braking system).

SO IS THE COSYFIT HI-LO OR COSYFIT HI-LO (PROFILING) SUITABLE FOR YOU?

Due to the wide variety of needs and behaviours in special needs clients, if you are thinking of ordering a Cosyfit Hi-Lo or Cosyfit Hi-Lo (Profiling) we will discuss your individual circumstances with you over the phone. We will discuss your client/s's needs, mobility, behaviour and size in order to advise.

You can also see Carer Reviews on our website www.safespaces.co.uk. These give you an idea of how the these products have worked for others in real situations, both at home and in care situations, and may help you assess whether it is suitable for your needs. You can also see more images of them on the website.

SPECIAL CIRCUMSTANCES

We specialise in trying to offer custom-made solutions for clients with a range of sometimes unusual and/or demanding needs and circumstances. Modifications can be made to the specifications of the Cosyfit Hi-Lo and Cosyfit Hi-Lo (Profiling) in special circumstances. Please contact us to discuss your individual requirements and we will see if we can help.

PLEASE CONTACT US FOR CURRENT PRICES AND A QUOTE



Safespaces (Cornholme) Ltd, 11 Cleveland St, Todmorden, Lancs, OL14 8LZ

Tel/Fax: 01706 816274 - email: info@safespaces.co.uk

www.safespaces.co.uk